### The problem Dry, irritated, itchy eyes

Let your doctor know if you have any of the following symptoms:

- Dry, itchy or burning eyes
- Watery, gritty or blurry eyes
- Sensitivity to light, causing you to squint or blink
- Difficulty focusing due to dryness

## Dryness is caused by a variety of factors:


- Age
- Eve surgery
- Extended computer use
- Contact lens wear
- Certain medications
- Allergies
- Eve disorders
- Diseases (Arthritis, Diabetes, etc.)
- Hormonal imbalance
- Environmental factors

Contact us:

(888) 827-8337

# A simple & natural solution

The **Bruder Moist Heat Eye Compress** delivers a doctor-recommended moist heat treatment in just minutes from the microwave.

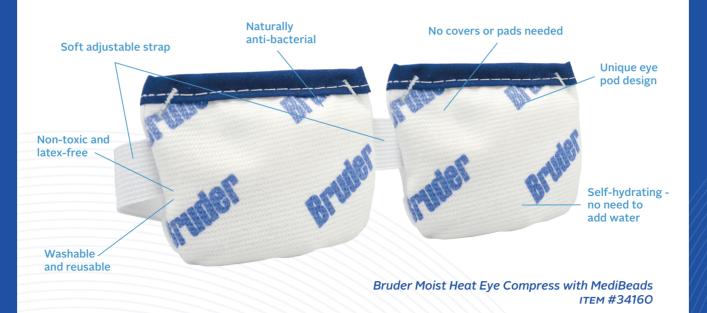

Ask your doctor why adding the easy-to-use **Bruder Moist Heat Eye Compress** to your eye wellness routine will help relieve your symptoms safely, effectively, and economically.

# The solution Bruder Moist Heat Eve Compress

The compress relieves dryness simply, naturally, and conveniently. Just place the compress in a clean microwave for 20-30 seconds, then apply over closed eyes for 10 minutes. The **moist heat** provides welcome relief in minutes. The compress will maintain prescribed heat levels for the entire treatment. Use as often as necessary, typically one or two times a day.

The **Bruder Moist Heat Eye Compress** will not interfere with other prescriptions or over-the-counter medications. It can be hand washed when needed and will last 6-9 months with frequent daily use.

#### How it works:

The Bruder Moist Heat Eye Compress features patented MediBeads® technology. Water molecules are continuously absorbed from the air and stored inside the beads. When microwaved, the absorbed water is released as clean moist heat. There is no wait, no mess, and no need to add water.

The moist heat provided by the Bruder Moist Heat Eye Compress may also be a perfect solution for treating Blepharitis, Styes, and Chalazions.